

Accu-Chek Aviva. Security feels good.



For further instructions on handling please watch the product video at www.accu-chek.de/aviva or the package insert.

Testing for blood glucose



1 Insert a test strip

Insert the test strip into the meter in the direction of the arrow until you hear a signal. A blood drop symbol flashes.



2 Taking a blood sample

Obtain a drop of blood from the side of your fingertip using the lancet (brief instructions on back).



3 Applying blood

Touch the blood drop with the end of the test strip. Do not put blood on the top of the test strip. The test strip has absorbed sufficient blood as soon as the signal sounds and the hour glass starts to flash.



4 Reading the value

The value appears on the display after 5 seconds. The meter turns itself off after the test strip is removed.


Accu-Chek FastClix. Obtain blood in just 1 click.

Preparation



Adjusting penetration depth


Rotate cap until the marking shows the desired penetration depth

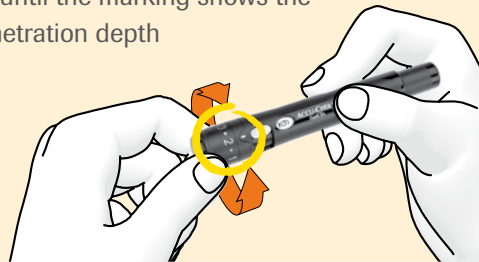

Obtaining blood: just 1 click!

Taking a blood sample
Press plunger down completely



Just 1 click
for priming and triggering

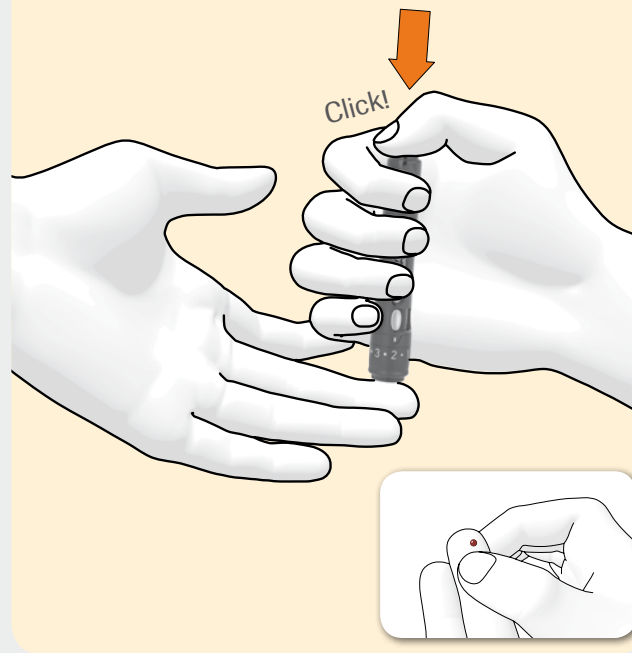

Next step



Load a new lancet

Turn lever in direction A to the stop and then back in direction B.

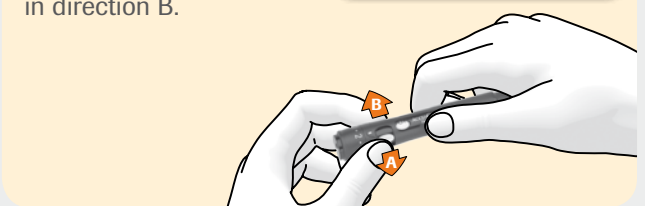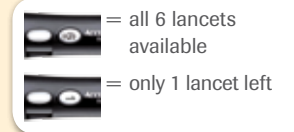

Remove used lancet drum

Remove cap and pull out the drum

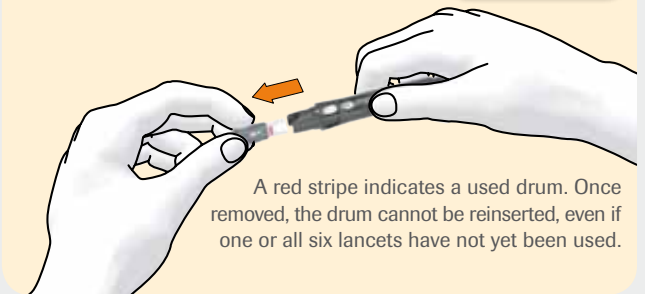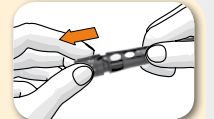

ACCU-CHEK, ACCU-CHEK AVIVA and FASTCLIX are Roche trademarks.
© 2016 Roche Diabetes Care. All rights reserved.

You would like to know more?

Our staff at the Accu-Chek Customer Service Center are at your service!
Toll-free number 0800 4466800 (Mon-Fri, 08:00-18:00 hrs).
Or visit us online at www.accu-chek.de



www.accu-chek.de
Roche Diabetes Care Deutschland GmbH
Sandhofer Straße 116
68305 Mannheim

ACCU-CHEK®