

# ACCU-CHEK® Compact Plus

## Testing your blood glucose

Accu-Chek Compact Plus comes with batteries pre-installed, and the date and time pre-set. Accu-Chek Compact Plus will automatically code on insertion of the test strip drum.

### Inserting or replacing a test strip drum



1 Open drum compartment cover.



2 Press the red drum release button.



3 Tip the meter forwards, the test strip drum drops out.



4 Insert new test strip drum.


### Testing blood glucose



1 Pull down the test button to advance a test strip.



2 Hold the meter so the test strip is pointing downwards. Touch the drop of blood against the front edge of the test strip where the black notch is located. The test strip draws up blood.

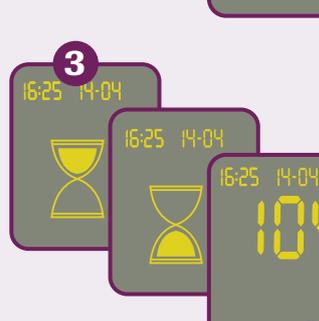

3 See display for meter reading.



4 Pull down the test button to eject the used test strip.

# ACCU-CHEK® *Softclix Plus*

The Accu-Chek Softclix Plus lancing device can be detached from the Accu-Chek Compact Plus meter using the slide button on the back of the meter, or used whilst still attached.



**1** Remove the cap from the lancing device.



**5** Press the cap back on to the lancing device. The cap snaps in to place with a click.



**2** Press the plunger at the top of the lancing device to eject the used lancet.



**6** Twist the cap to the desired depth setting (11 settings in total).



**3** Slide a new lancet in to the lancet holder and push it in until it clicks in to place.



**7** Press the plunger all the way down. The plunger returns about a third of the way. The lancing device is now primed.



**4** Remove the protective cap from the lancet with a twisting motion.



**8** Place the lancing device against the chosen test site, press the plunger at the top of the lancing device down again until it clicks to obtain a sample.

The Accu-Chek Softclix Plus lancing device is intended for patient self-monitoring by a single person. It must not be used to collect blood in a multi-patient setting as it does not incorporate any features to guard against cross-infection.

This Handling Chart does not replace the comprehensive Reference Manual for your Accu-Chek Compact Plus blood glucose system. Please be sure to read the safety notices contained in the Reference Manual.

ACCU-CHEK, SOFTCLIX and LIVE LIFE. THE WAY YOU WANT. are trademarks of Roche.  
© 2007 Roche Diagnostics

[www.accu-check.com](http://www.accu-check.com)  
Roche Diagnostics GmbH  
D-68298 Mannheim



## ACCU-CHEK®

Live life. The way you want.