

Testing for blood glucose



1

Removing the test strip


Push a test strip upwards with slight pressure from the front or back. With this test strip you can also touch the upper side.



2

Inserting a test strip

Insert the test strip into the test strip slot in the direction of the arrow. If you switch on the device beforehand, you will automatically activate illumination.

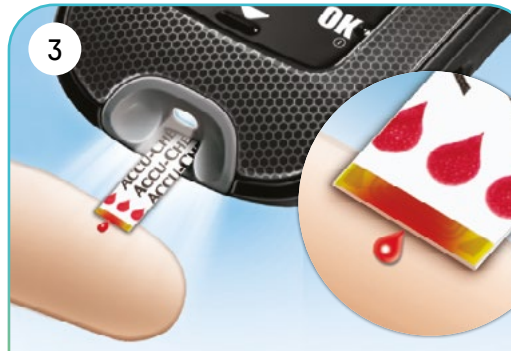

3

Applying blood

Touch the drop of blood with the yellow edge of the test strip and wait until the "Measurement in progress" message appears.



4

Ejecting the test strip

Remove the test strip by activating the test strip ejector button or by pulling the test strip out.

Preparation

Insert lancet drum



- 1 Remove cap
- 2 Insert drum
- 3 Replace cap



Rotate the cap until the marking shows the desired penetration depth.

Obtaining blood

Just 1 click!

Only
1 click
for priming and
triggering



Press release button down completely to take a blood sample.

Next step

Preparing and removing the lancet drum



- 1 First turn lever downwards to stop, then back in upwards direction.
- 2 For removal, pull off cap and pull out the drum.

Unused drum



Used drum



Attention: do not remove the lancet drum until all the lancets have been used. Once removed, the drum cannot be reinserted for hygienic reasons.