

ACCU-CHEK® MOBILE


MEASURE YOUR BLOOD GLUCOSE WHENEVER AND WHEREVER YOU WANT.

MEASURING IN 4 EASY STEPS



1

Opening
Open the tip protector.
The measuring device switches on.

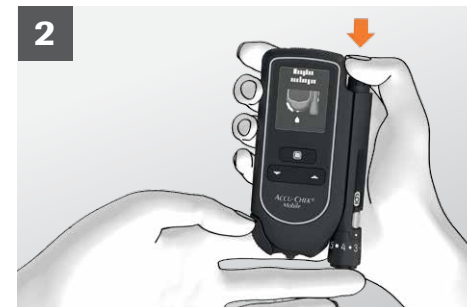

2

Taking a blood sample
Press the lancing device firmly against the side of your finger tip. Now press the release button down all the way; only 1 click for setting and release.



3

Measuring
Apply the drop of blood to the center of the test field. Only touch the test field lightly.



4

Closing
You can read the result after approximately 5 seconds. When you close the tip protector, the measuring device switches off automatically.




SET-UP / CHANGING THE TEST CARTRIDGE (1-3)



1 Open the tip protector to the stop. Only then can you open the cartridge compartment and insert the test cartridge.



2 Push the locking button of the cartridge compartment lid upwards.

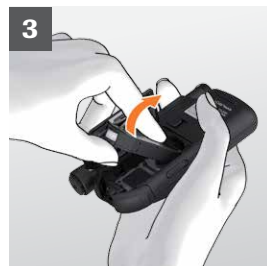

3 Insert a new test cartridge into the measuring device. Close the lid of the cartridge compartment and the tip protector.



4 Pull the protective foil for the batteries from the measuring device in backward direction. Pull the protective foil from the display.

PREPARATION OF A LANCET



1 Prepare the lancing device for the next blood sample: move the lever in direction ① and back again in direction ②.



2 Rotate the cap, until the marking shows the desired penetration depth. Start with a medium penetration depth, i.e. 3.



3 Only remove the lancet drum after all 6 lancets have been used.



Once a lancet drum has been removed, it cannot be re-inserted. This is indicated by the red stripe.

CHANGING THE LANCET DRUM



1 Pull the cap from the lancing device.



2 Remove the lancet drum.



3 Insert the new lancet drum (white end first). You now have 6 lancets available.



4 Replace the cap.



www.accu-chek.de
Roche Diabetes Care Deutschland GmbH
Sandhofer Straße 116
68305 Mannheim

ACCU-CHEK, ACCU-CHEK MOBILE and FASTCLIX are trademarks of Roche.
© 2019 Roche Diabetes Care. All rights reserved.