ACCU-CHEK and ACCU-CHEK FLEXLINK
are trademarks of Roche.

Roche Diagnostics GmbH
Sandhofer Strasse 116
68305 Mannheim, Germany
www.accu-chek.com

04537246001/E

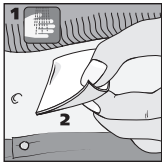
Status: 04-2013

ACCU-CHEK[®] *FlexLink*


INFUSION SET


1


2

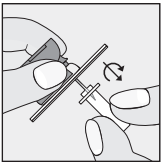
3

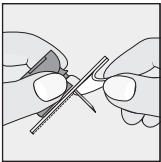
4

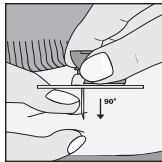
5

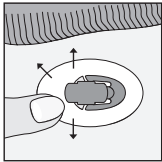
6

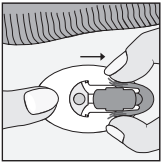7

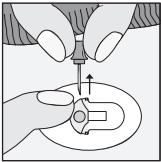8

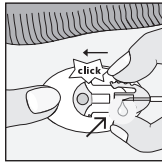Instructions for use	4	Návod k použití	88
Gebrauchsanweisung	16	Instrukcja użytkowania	100
Mode d'emploi	28	Brugsanvisning	112
Istruzioni per l'uso	40	Bruksanvisning	122
Instrucciones de uso	52	Bruksanvisning	134
Gebruiksaanwijzing	64	Käyttöohjeet	144
Instruções de uso	76		

Instructions for use
Gebrauchsanweisung
Mode d'emploi
Istruzioni per l'uso
Instrucciones de uso
Gebruiksaanwijzing
Instruções de uso
Návod k použití
Instrukcja użytkowania
Brugsanvisning
Bruksanvisning
Bruksanvisning
Käyttöohjeet

en

de

fr

it

es

nl

pt

cs

pl

da

sv

no

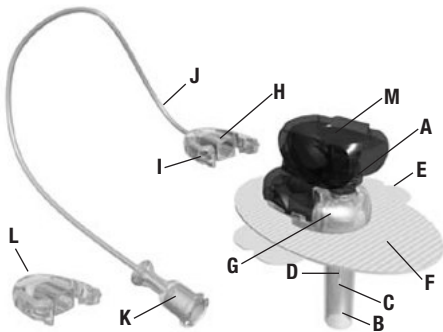
fi

ACCU-CHEK[®] FlexLink

INFUSION SET

transfer-set

head-set

Indications for use

Accu-Chek FlexLink is an infusion set for the subcutaneous infusion of insulin administered with microdosage insulin pumps.

Contents of the Accu-Chek FlexLink packages

Package 1 head-set and transfer-set

- A** introducer needle handle
- B** protective cover
- C** introducer needle
- D** soft cannula
- E** adhesive backing
- F** self-adhesive

- G** cannula housing
- H** connector
- I** side clips of the connector
- J** tubing
- K** luer-lock
- L** disconnect cover
- M** blue holder cap

Package 2 head-set

Components A–G, L and M from package 1.

May only be used together with the transfer-set from package 1.

⚠ WARNINGS

Read the instructions carefully before use. It is recommended that you consult your insulin pump manual for information regarding pump therapy.

For first time use, setup must be performed in the presence of a health care professional.

- This device remains sterile until the package is opened or damaged. Do not use if the sterile package has been opened or damaged.
- Protect the product from excessive humidity, sunlight and heat. Store at room temperature.
- Discard the set after each use. Do not clean or re-sterilize.

- The Accu-Chek FlexLink is for subcutaneous use only.
- Always tighten the luer-lock firmly into the pump adapter. If this is not done, danger of leakage can result, due to a loose connection between the infusion set and the pump. Avoid using any tools to screw the luer-lock connector into place. Otherwise damage could result to the luer-lock connector.
- Do not leave air bubbles in the Accu-Chek FlexLink. Prime thoroughly.
- A small percentage of Accu-Chek FlexLink using soft cannulas can accidentally get crimped during insertion or displaced while in use. If this should occur you should immediately

use a new Accu-Chek FlexLink.

- Check the infusion site frequently to make sure the soft cannula remains firmly in place. Because the cannula is so soft, you will not notice any pain if it is pulled out. The soft cannula must always be completely inserted in order to receive the full amount of insulin. Replace the Accu-Chek FlexLink as soon as the self-adhesive loosens.
- Do not reinsert the introducer needle into the soft cannula. Reinsertion could cause tearing of the soft cannula which would result in unpredictable medication flow.
- If you have to disconnect your Accu-Chek FlexLink pay particular attention to hygiene.

- Never use a soft cannula for more than 72 hours (3 days). Never use the tubing for more than 6 days. When changing the cartridge, make sure that the transfer-set is completely primed before connecting to the head-set.
- If the infusion site becomes inflamed, replace your Accu-Chek FlexLink and use a new site.
- Do not put disinfectants, perfumes, deodorants or insect repellent on your Accu-Chek FlexLink as these may affect the integrity of the set.
- Never prime the transfer-set or attempt to free a clogged line while the transfer-set is connected to the head-set. You may accidentally infuse an uncontrolled quantity of insulin.

- Avoid mechanical stress on the infusion site and on the Accu-Chek FlexLink. Pay attention when carrying heavy weights.
- Before inserting the Accu-Chek FlexLink, prepare the infusion site in accordance with the instructions of your healthcare professional.
- Check blood glucose level 1–3 hours after inserting your Accu-Chek FlexLink and inspect the infusion site on a regular basis. Do not insert a new soft cannula just prior to bedtime. Check blood glucose frequently (check with your physician).
- Check your infusion set at the luer-lock to tube connection periodically (every 3 hours during

the day and before going to sleep) during use. A break may result in leakage, causing interruption of insulin delivery. If you notice a break, change your infusion set immediately and check your blood glucose.

- If blood glucose level gets unexplainably too high, or an «occlusion» alarm occurs, check for clogs and leaks. If in doubt, change your Accu-Chek FlexLink.

Insert Accu-Chek FlexLink

Picture 1

Choose a site away from the waistline, bones, scar tissue, belly button and recent infusion sites. Clean hands and the site according to the instructions from your health care professional. Allow infusion site to dry before the soft cannula is inserted.

Picture 2

While holding the Accu-Chek FlexLink at the blue holder cap (M) with thumb and forefinger, remove the transparent protective cover (B) by turning and pulling. Keep the protective cover for later use in order to safely dispose the introducer needle (C).

Picture 3

Remove both pieces of the adhesive backing (E) from the self-adhesive (F).

Picture 4

Pinch the subcutaneous tissue. Insert the introducer needle (C) and soft cannula (D) quickly and straight into the infusion site, at a 90° angle. Watch the soft cannula while inserting to assure that it goes in smoothly without kinking or crimping.

⚠ WARNING

Ensure clean application. Do not touch the soft cannula or introducer needle. Make sure that the soft cannula is fully inserted into the subcutaneous tissue and not kinked at the skin surface.

Once the introducer needle punctures the skin, the soft cannula must be quickly pushed in the rest of the way, or it may kink upon insertion, causing reduced or blocked insulin delivery.

Picture 5

Carefully smooth out the self-adhesive (F) onto the skin and run your fingers over the self-adhesive to ensure good skin contact is achieved.

Picture 6

Hold the self-adhesive (F) in place, gently press the side clips of the blue holder cap (M) and pull it from the cannula housing (G).

Picture 7

To remove the introducer needle (C) hold the cannula housing (G) firmly to your body. Place one finger on the top of the cannula housing and withdraw the introducer needle in a straight manner using the introducer needle handle (A). Remember to dispose the introducer needle (C) safely (needle container). Discard your introducer needle according to local regulations. If you inserted the infusion set all by yourself and no safe needle container is at hand, consider using the protective cover to cover the needle. Ensure to avoid needle injury of others with the risk of infection.

Picture 8

Attach the luer-lock (K) to the pump by tightening it into the pump adapter. Follow the pump manufacturer's user manual to prime the transfer-set. Prime the transfer-set until medication drips from the connector needle tip. Do not leave air bubbles in either the cartridge or the transfer-set. Connect the transfer-set to the newly inserted head-set and fill the empty space of the cannula housing with a bolus of 1.0 units (U100 insulin).

WARNING

You must fill the empty space of the newly inserted head-set after connecting the transfer-set. Failure to do so results in missed insulin.

If the coupling does not click noticeably and audibly together, use a new head-set and a new transfer-set.

Do not use any tools to screw the luer-lock connector into place.

Filling volume transfer-set:

12"/ 30 cm: approx. 6 units* approx. 60 μ l

24"/ 60 cm: approx. 11 units* approx. 110 μ l

31"/ 80 cm: approx. 14 units* approx. 140 μ l

43"/110 cm: approx. 18 units* approx. 180 μ l

Filling volume head-set:

approx. 1.0 units* approx. 10 μ l

*U 100 Insulin

WARNING

Change head-set every 2–3 days (max. 72 hours). Otherwise you risk insulin resistance or inflammation of the infusion site. Never use the transfer-set for more than 6 days.

Disconnect Accu-Chek FlexLink

WARNING

Please consult with your health care professional on how to compensate for any missed insulin while you are disconnected. Carefully monitor blood glucose level while disconnected from the pump and after you reconnect.

Accu-Chek FlexLink allows you to temporarily disconnect from your pump for showering, swimming, etc., without changing your Accu-Chek FlexLink.

Hold the self-adhesive (F) in place, gently press the side clips of the connector (I) and pull the

connector (H) from the cannula housing (G). Put the disconnect cover (L) on the cannula housing.

Reconnect Accu-Chek FlexLink

WARNING

Before reconnecting, ensure that a drop of medication appears at the tip of the connector needle.

Fill the Accu-Chek FlexLink until medication comes out of the connector needle. Hold the self-adhesive (F) in place and remove the disconnect cover (L) from the cannula housing (G). Reattach the connector (H) to the cannula housing. An audible «click» ensures the connector is locked.

WARNING

It is not necessary to give an additional bolus.

Distributed in the United Kingdom by:

Roche Diagnostics Limited
Charles Avenue, Burgess Hill,
West Sussex, RH15 9RY, United Kingdom

Accu-Chek Pump Care Line:

UK Freephone number: 0800 731 22 91*

ROI Freephone number: 1800 88 23 51*

burgesshill.insulinpumps@roche.com

www.accu-chek.co.uk

www.accu-chek.ie

*Calls may be recorded for training purposes

Distributed in Australia by:

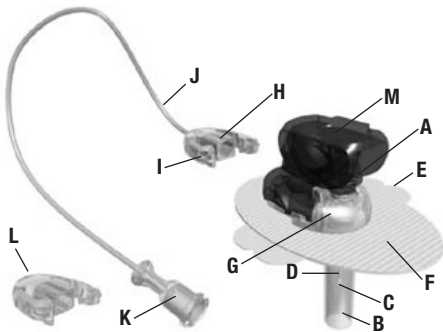
Roche Diagnostics Australia Pty. Ltd.
31 Victoria Avenue, Castle Hill,
NSW 2154, Australia
ABN 29 003 001 205
Phone: +61 1800 633 457 (Pump Hotline)
Phone: +61 1800 802 409 (Orders)
Fax: +61 1800 066 598 (Orders)
www.accu-chek.com.au

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