Made in Denmark

ACCU-CHEK and ACCU-CHEK TENDERLINK
are trademarks of Roche.

Distributed by:

Disetronic Medical Systems AG
3401 Burgdorf, Switzerland

Unomedical a/s
4000 Roskilde, Denmark

03692051001/V03/07.07

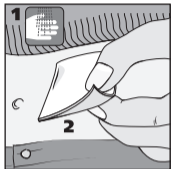
ACCU-CHEK[®]

TenderLink


INFUSION SET


1


2

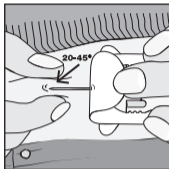
3

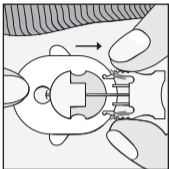
4

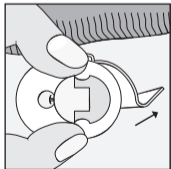5

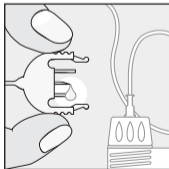6

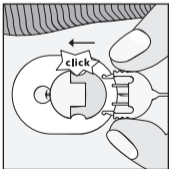
Instructions for use	4	Kasutusjuhised	224
Gebrauchsanweisung	20	הוראות שימוש	240
Mode d'emploi	36	Upute za uporabu	256
Istruzioni per l'uso	52	Használati útmutató	272
Instrucciones de uso	68	Naudojimo instrukcijos	288
Gebbruiksaanwijzing	84	Lietošanas instrukcija	304
Instruções de uso	100	Instrucţiuni de utilizare	320
Návod k použití	116	Инструкции по использованию	336
Instrukcja użytkowania	130	Návod na používanie	352
Bruksanvisning	146	Navodila za uporabo	368
Bruksanvisning	162	Uputstvo za upotrebu	384
Käyttöohjeet	176	Kullanım talimatları	398
Brugsanvisning	190	使用说明	414
Οδηγίες χρήσης	206	使用說明	428

Instructions for use	en
Gebrauchsanweisung	de
Mode d'emploi	fr
Istruzioni per l'uso	it
Instrucciones de uso	es
Gebruiksaanwijzing	nl
Instruções de uso	pt
Návod k použití	cs
Instrukcja użytkowania	pl
Bruksanvisning	sv
Bruksanvisning	no
Käyttöohjeet	fi
Brugsanvisning	da
Οδηγίες χρήσης	el


ACCU-CHEK® TenderLink

INFUSION SET

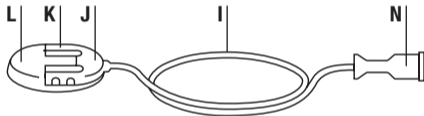transfer set

head set

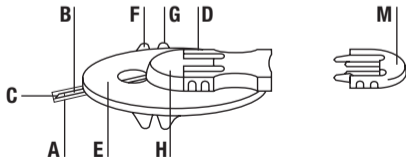Description

Accu-Chek TenderLink is an infusion set for the subcutaneous infusion of insulin administered with microdosage insulin pumps.

Contents of the package Accu-Chek TenderLink II

Inner packages 1 and 2.

Package 1 (complete set):

- | | |
|---|--|
| A protective cover | G back side of the adhesive backing |
| B soft cannula | H cannula housing |
| C introducer needle | I tubing |
| D side clips of the introducer needle | J connector with connector needle |
| E self-adhesive | K side clips of the connector |
| F front half of the the adhesive backing | L circular protective cap |
| | M disconnect cover |
| | N luer-lock |

Package 2 (additional needle head): Components A–H from package 1.

Contents of the package Accu-Chek TenderLink I

Package 1 only.

Warning

Read the instructions carefully before use.

It is recommended that you consult your insulin pump manual for information regarding pump therapy.

For first time use, setup must be performed in the presence of a health care professional.

- This device remains sterile until the package is opened or damaged. Do not use if the sterile package has been opened or damaged.
- Protect the product from excessive humidity, sunlight and heat. Store at room temperature.

- Discard the set after each use. Do not clean or re-sterilize.
- The Accu-Chek TenderLink is for subcutaneous use only.
- Always tighten the luer-lock firmly into the pump adapter. If this is not done, danger of leakage can result, due to a loose connection between the infusion set and the pump. Avoid using any tools to screw the luer-lock connector into place. Otherwise damage could result to the luer-lock connector.
- Do not leave air bubbles in the Accu-Chek TenderLink. Prime thoroughly.
- A small percentage of Accu-Chek TenderLink using soft cannulas can accidentally get crimped during insertion or displaced while in use. If this should occur you should immediately use a new Accu-Chek TenderLink.
- Check frequently to make sure the soft cannula remains firmly in place. Because the cannula is so soft, you will not notice any pain if it is pulled out. The soft cannula must always be completely inserted in order to receive the full amount of medication. Replace the Accu-Chek TenderLink as soon as the self-adhesive loosens.

- Do not reinsert the introducer needle into the soft cannula. Reinsertion could cause tearing of the soft cannula which would result in unpredictable medication flow.
- If you have to disconnect your Accu-Chek TenderLink, pay particular attention to hygiene.
- Never use a soft cannula for more than 72 hours (3 days). Never use the transfer set for more than 6 days. When changing the cartridge, make sure that the transfer-set is completely primed before connecting to the head-set.
- If the infusion site becomes inflamed, replace the Accu-Chek TenderLink and use a new site.
- Do not put disinfectants, perfumes, deodorants or insect repellent on your Accu-Chek TenderLink as these may affect the integrity of the set.
- Never prime the transfer set or attempt to free a clogged line while the transfer set is connected to the head-set. You may accidentally infuse an uncontrolled quantity of insulin.

- Avoid mechanical stress on the infusion site and on the Accu-Chek TenderLink. Pay attention when carrying heavy weights.
- Before inserting the Accu-Chek TenderLink, prepare the infusion site in accordance with the instructions of your healthcare professional.
- Check blood glucose level 1–3 hours after inserting your Accu-Chek TenderLink and inspect the infusion site on a regular basis. Do not insert a new soft cannula just prior to bedtime. Check blood glucose frequently (check with your physician).
- If blood glucose level gets unexplainably too high, or an “occlusion” alarm occurs, check for clogs and leaks. If in doubt, change your Accu-Chek TenderLink.

Insert Accu-Chek TenderLink

Picture 1

Choose a site away from the waistline, bones, scar tissue, belly button and recent infusion sites. Clean hands and the site according to the instructions from your health care professional. Allow infusion site to dry before the soft cannula is inserted.

Remove the front half of the adhesive backing (F) from the self-adhesive (E) and pull the protective cover (A) off the introducer needle (C). Grasp the Accu-Chek TenderLink between the thumb and middle finger of one hand behind the side clips and pull back the edge of the self-adhesive with the index finger.

Warning

Ensure clean application. Do not touch the soft cannula or the introducer needle.

Picture 2

With your free hand, pinch together a bit of subcutaneous tissue at the disinfected site. Insert the introducer needle (C) under the skin at an angle of 20–45°, proceeding quickly and in one motion. Watch the soft cannula (B) while inserting to assure that it goes in smoothly without kinking or crimping.

⚠ Warning

People with very small or large amounts of subcutaneous fatty tissue should take care when choosing the angle of insertion or the cannula length. The cannula could be lodged in the muscular tissue or epidermis, where absorption of the medication is limited or impossible. Ask the doctor who is treating you about this. Make sure that the soft cannula is fully inserted into the subcutaneous tissue and not kinked at the skin surface.

Once the introducer needle punctures the skin, the soft cannula must be quickly pushed in the rest of the way, or it may kink upon insertion, causing reduced or blocked medication delivery.

Picture 3

Press the self-adhesive (E) firmly onto the skin in order to ensure secure seating of Accu-Chek TenderLink. Place one finger in front of the inspection window on the Accu-Chek TenderLink to fix it. Press the side clips of the introducer needle (D) together gently and at the same time pull the introducer needle (C) out of the skin.

Warning

Dispose of the introducer needle safely, so that nobody can prick or injure themselves on it.

Picture 4

Remove the back side of the adhesive backing (G) from the self-adhesive (E) and press the self-adhesive firmly onto the skin.

Picture 5

Attach the luer-lock (N) to the pump by tightening it into the pump adapter. Make sure there is a cartridge containing sufficient insulin in the cartridge compartment. Remove the circular protective cap (L) by pressing the side clips inwards gently. Follow the pump manufacturer's user manual to prime the transfer-set until medication drips from the connector needle tip. Do not leave air bubbles in either the cartridge or the transfer-set.

Picture 6

Connect the transfer-set to the newly inserted head-set with the bulging side upwards. A click must be heard when the connection engages. Fill the empty space of the cannula housing (H) with a bolus of 0.7 units of insulin U-100 (about 7µl). Start distribution with the pump.

Warning

Change the head-set every 2–3 days (max. 72 hours). Otherwise you risk insulin resistance or inflammation of the infusion site.

Never use the tube-set for more than 6 days.

You must fill the empty space of the newly inserted head-set after connecting the transfer set. Failure to do so results in missed insulin.

Always check your blood glucose level 1–3 hours after introducing the set.

If the coupling does not click noticeably and audibly together, use a new head set and a new transfer set.

Do not use any tools to screw the luer-lock connector into place.

Filling volume:

30 cm/12"	approx 7 units *	approx 70 µl
60 cm/24"	approx 10 units *	approx 100 µl
80 cm/31"	approx 13 units *	approx 130 µl
110 cm/43"	approx 18 units *	approx 180 µl
*U100 insulin		

Disconnect Accu-Chek TenderLink

Accu-Chek TenderLink allows you to temporarily disconnect from your pump for showering, swimming, etc., without changing your Accu-Chek TenderLink.

Warning

Please consult with your health care professional on how to compensate for any missed insulin while you are disconnected. Carefully monitor blood glucose level while disconnected from the pump and after you reconnect.

Step 1

Stop the pump. Hold the self-adhesive (E) in place, gently press the side clips of the connector (K) and pull the connector (J) from the cannula housing (H).

Step 2

Put the disconnect cover (M) on the cannula housing.

Reconnect Accu-Chek TenderLink

Warning

Before reconnecting, ensure that a drop of medication appears at the tip of the connector needle.

Step 1

Fill the Accu-Chek TenderLink until medication comes out of the connector needle.

Step 2

Hold the self-adhesive (E) in place and remove the disconnect cover (M) from the cannula housing (H).

Reattach the connector (J) to the cannula housing. An audible “click” ensures the connector is locked.

⚠ Warning

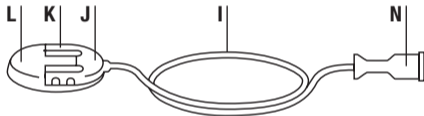
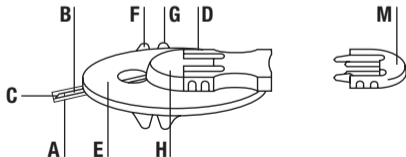
It is not necessary to give an additional bolus.

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